
Field Safety Notice

ENFit Nasogastric Feeding Tubes

Ref: FSN-006

September 2021

Immediate Attention: All Nursing, Medical and Technical staff using these devices.

Device Description

The HMC and Medicina Nasogastric and Nasojejunal range of tubes are used by healthcare professionals across a wide variety of patients from neonates to adults, both in hospital and at home, to feed, flush, deliver medicines and to aspirate stomach content to confirm correct placement.

Reason for Safety Notice/ Explicit Contraindications of the devices

This notice is issued to ensure clarification on the correct use of the Nasogastric and Nasojejunal tube ranges.

Both HMC and Medicina nasogastric and Nasojejunal feeding tubes are not intended and should never be used for high volume decompression in emergency situations or decompression for high volume or high viscous fluid.

This is due to the inner diameter of ENFit connector (as per ISO 80369-3 standard), which is used on Nasogastric and Nasojejunal feeding tubes with the specific intended use of enteral formula delivery, and not for high volume decompression in emergency situations or decompression for high volume or high viscous fluid.

The full standard name is detailed below for your information:

ISO 80369-3:2016 Small-bore connectors for liquids and gases in healthcare applications — Part 3: Connectors for enteral applications

Where quick decompression of high amounts of stomach content and/or the connection to a suction are required, large bore decompression tubes with a suitable connector (e.g. Funnel connector) should be considered.



Leaders in enteral delivery

Affected Medicina Product codes

Code	Product Description
NGP10/120C	Purple F/Bore NG 10FR 120cm Poly- ENFit (NGP10/120C)
NGP10/120CW	Purple F/Bore NG 10FR 120cm W/Poly-ENFit (NGP10/120CW)
NGP10/85C	Purple F/Bore NG 10FR 85cm Poly- ENFit (NGP10/85C)
NGP10/85CW	Purple F/Bore NG 10FR 85cm W/Poly -ENFit (NGP10/85CW)
NGP10/95YC	Purple F/Bore NG 10FR 95cm Poly With Dual Port- ENFit (NGP10/98YC)
NGP12/120C	Purple F/Bore NG 12FR 120cm Poly - ENFit (NGP12/120C)
NGP12/120CW	Purple F/Bore NG 12FR 120cm W/Poly -ENFit (NGP12/120CW)
NGP12/85C	Purple F/Bore NG 12FR 85cm Poly- ENFit (NGP12/85C)
NGP12/85CW	Purple F/Bore NG 12FR 85cm W/Poly- ENFit (NGP12/85CW)
NGP14/120C	Purple F/Bore NG 14FR 120cm Poly- ENFit (NGP14/120C)
NGP14/120CW	Purple F/Bore NG 14FR 120cm W/Poly- ENFit (NGP14/120CW)
NGP14/85C	Purple F/Bore NG 14FR 85cm Poly- ENFit (NGP14/85C)
NGP14/95YC	Purple F/Bore NG 14FR 95cm Poly With Dual Port- ENFit (NGP14/95YC)
NGP16/120C	Purple F/Bore NG 16FR 120cm W/Poly- ENFit (NGP16/120C)
NGP6/55C	Purple F/Bore NG 6FR 55cm Poly- ENFit (NGP6/55C)
NGP6/75C	Purple F/Bore NG 6FR 75cm Poly- ENFit (NGP6/75C)
NGP6/85C	Purple F/Bore NG 6FR 85cm Poly- ENFit (NGP6/85C)
NGP6/95YC	Purple F/Bore NG 6FR 95cm Poly With Dual Port- ENFit (NGP6/95YC)
NGP8/120C	Purple F/Bore NG 8FR 120cm Poly- ENFit (NGP8/120C)
NGP8/120CW	Purple F/Bore NG 8FR 120cm W/Poly- ENFit (NGP8/120CW)
NGP8/55C	Purple F/Bore NG 8FR 55cm Poly- ENFit (NGP8/55C)
NGP8/75C	Purple F/Bore NG 8FR 75cm Poly- ENFit (NGP8/75C)
NGP8/85C	Purple F/Bore NG 8FR 85cm Poly- ENFit (NGP8/85C)
NGP8/85CW	Purple F/Bore NG 8FR 85cm W/Poly- ENFit (NGP8/85CW)
NGP8/95YC	Purple F/Bore NG 8FR 95cm Poly With Dual Port- ENFit (NGP8/95YC)
CG10/120	Infant F/Tube Poly 10FR 120cm- ENFit (CG10/120)
CG10/160	Infant F/Tube Poly 10FR 160cm- ENFit (CG10/160)
CG10/80	Infant F/Tube Poly 10FR 80cm- ENFit (CG10/80)
CG10/80W	ENFit Infant Feeding Tube Polyurethane 10fr 80cm Weighted
CG12/120	Infant F/Tube 12FR 120cm Poly- ENFit (CG12/120)
CG12/80	Infant F/Tube 12FR 80cm Poly- ENFit (CG12/80)
CG12/80W	Infant F/Tube 12FR 80cm W/Poly- ENFit (CG12/80W)

Registered Office

Units 2-4, Rivington View Business Park, Station Road, Blackrod, Bolton BL6 5BN
Tel: 01204 695050 Fax: 01204 697755 email: info@medicina.co.uk www.medicina.co.uk

Medicina Ltd. A Company registered in England and Wales Reg. No. 02654669



Leaders in enteral delivery

Code	Product Description
CG14/80	Infant F/Tube 14FR 80cm Poly- ENFit (CG14/80)
CG4/120	Infant F/Tube Poly 4FR 120cm- ENFit (CG4/120)
CG4/160	Infant F/Tube Poly 4FR 160cm- ENFit (CG4/160)
CG4/40	Infant F/Tube Poly 4FR 40cm- ENFit (CG4/40)
CG4/50	Infant F/Tube Poly 4FR 50cm- ENFit (CG4/50)
CG4/50W	Infant F/Tube 4FR 50cm W/Poly- ENFit (CG4/50W)
CG4/80	Infant F/Tube Poly 4FR 80cm- ENFit (CG4/80W)
CG4/90	Infant F/Tube Poly 4FR 90cm- ENFit (CG4/90)
CG5/40	Infant F/Tube Poly 5FR 40cm- ENFit (CG5/40)
CG5/50	Infant F/Tube Poly 5FR 50cm- ENFit (CG5/50)
CG5/80	Infant F/Tube Poly 5FR 80cm- ENFit (CG5/80)
CG6/100	Infant F/Tube Poly 6FR 100cm- ENFit (CG6/100)
CG6/120	Infant F/Tube Poly 6FR 120cm- ENFit (CG6/120)
CG6/160	Infant F/Tube Poly 6FR 160cm- ENFit (CG6/160)
CG6/40	Infant F/Tube Poly 6FR 40cm- ENFit (CG6/40)
CG6/50	Infant F/Tube Poly 6FR 50cm- ENFit (CG6/50)
CG6/50W	Infant F/Tube 6FR 50cm W/Poly- ENFit (CG6/50W)
CG6/80	Infant F/Tube Poly 6FR 80cm- ENFit (CG6/80)
CG6/80W	Infant F/Tube 6FR 80cm W/Poly- ENFit (CG6/80W)
CG6/90	Infant F/Tube Poly 6FR 90cm- ENFit (CG6/90)
CG8/120	Infant F/Tube Poly 8FR 120cm- ENFit (CG8/120)
CG8/40	Infant F/Tube Poly 8FR 40cm- ENFit (CG8/40)
CG8/50	Infant F/Tube Poly 8FR 50cm- ENFit (CG8/50)
CG8/80	Infant F/Tube Poly 8FR 80cm- ENFit (CG8/80)
CG8/80W	Infant F/Tube 8FR 80cm W/Poly- ENFit (CG8/80W)

Affected HMC product codes (Distributed by Medicina)

Code	Product Description
LG10/120	ENFit Nasogastric Feeding Tube10fr 120cm
LG10/160	ENFit Nasogastric Feeding Tube10fr 160cm
LG10/80	ENFit Nasogastric Feeding Tube10fr 80cm
LG10/80W	ENFit Nasogastric Feeding Tube10fr 80cm weighted
LG12/120	ENFit Nasogastric Feeding Tube12fr 120cm

Registered Office

Units 2-4, Rivington View Business Park, Station Road, Blackrod, Bolton BL6 5BN

Tel: 01204 695050 Fax: 01204 697755 email: info@medicina.co.uk www.medicina.co.uk

Medicina Ltd. A Company registered in England and Wales Reg. No. 02654669



Leaders in enteral delivery

Code	Product Description
LG12/80	ENFit Nasogastric Feeding Tube12fr 80cm
LG12/80W	ENFit Nasogastric Feeding Tube12fr 80cm Weighted
LG14/80	ENFit Nasogastric Feeding Tube14fr 80cm
LG16/120	ENFit Nasogastric Feeding Tube 16fr 120cm
LG18/120	ENFit Nasogastric Feeding Tube18fr 120cm
LG4/120	ENFit Nasogastric Feeding Tube4fr 120cm
LG4/125	ENFit Nasogastric Feeding Tube4fr 125cm
LG4/160	ENFit Nasogastric Feeding Tube4fr 160cm
LG4/40	ENFit Nasogastric Feeding Tube4fr 40cm
LG4/50	ENFit Nasogastric Feeding Tube4fr 50cm
LG4/80	ENFit Nasogastric Feeding Tube4fr 80cm
LG4/90	ENFit Nasogastric Feeding Tube4fr 90cm
LG5/40	ENFit Nasogastric Feeding Tube5fr 40cm
LG5/50	ENFit Nasogastric Feeding Tube5fr 50cm
LG5/80	ENFit Nasogastric Feeding Tube5fr 80cm
LG6/120	ENFit Nasogastric Feeding Tube6fr 120cm
LG6/160	ENFit Nasogastric Feeding Tube6fr 160cm
LG6/40	ENFit Nasogastric Feeding Tube6fr 40cm
LG6/50	ENFit Nasogastric Feeding Tube6fr 50cm
LG6/50W	ENFit Nasogastric Feeding Tube6fr 50cm Weighted
LG6/80	ENFit Nasogastric Feeding Tube6fr 80cm
LG6/80W	ENFit Nasogastric Feeding Tube6fr 80cm weighted
LG6/90	ENFit Nasogastric Feeding Tube6fr 90cm
LG8/120	ENFit Nasogastric Feeding Tube8fr 120cm
LG8/160	ENFit Nasogastric Feeding Tube8fr 160cm
LG8/40	ENFit Nasogastric Feeding Tube8fr 40cm
LG8/50	ENFit Nasogastric Feeding Tube8fr 50cm
LG8/50W	ENFit Nasogastric Feeding Tube8fr 50cm Weighted
LG8/80	ENFit Nasogastric Feeding Tube8fr 80cm
LG8/80W	ENFit Nasogastric Feeding Tube8fr 80cm weighted
LG8/90	ENFit Nasogastric Feeding Tube8fr 90cm
NGP10/120L	ENFit Nasogastric Feeding Tube 10fr 120cm
NGP10/85L	ENFit Nasogastric Feeding Tube 10fr 85cm

Registered Office

Units 2-4, Rivington View Business Park, Station Road, Blackrod, Bolton BL6 5BN
Tel: 01204 695050 Fax: 01204 697755 email: info@medicina.co.uk www.medicina.co.uk

Medicina Ltd. A Company registered in England and Wales Reg. No. 02654669



Leaders in enteral delivery

Code	Product Description
NGP10/85LW	ENFit Nasogastric Feeding Tube 10fr 85cm weighted
NGP12/100L	ENFit Nasogastric Feeding Tube 12fr 100cm
NGP12/120L	ENFit Nasogastric Feeding Tube 12fr 120cm
NGP12/85L	ENFit Nasogastric Feeding Tube 12fr 85cm
NGP14/120L	ENFit Nasogastric Feeding Tube 14fr 120cm
NGP14/85L	ENFit Nasogastric Feeding Tube 14fr 85cm
NGP16/120L	ENFit Nasogastric Feeding Tube 16fr 120cm
NGP5/55L	ENFit Nasogastric Feeding Tube 5fr 55cm
NGP6/55L	ENFit Nasogastric Feeding Tube 6fr 55cm
NGP6/75L	ENFit Nasogastric Feeding Tube 6fr 75cm
NGP6/85L	ENFit Nasogastric Feeding Tube 6fr 85cm
NGP8/120L	ENFit Nasogastric Feeding Tube 8fr 120cm
NGP8/55L	ENFit Nasogastric Feeding Tube 8fr 55cm
NGP8/75L	ENFit Nasogastric Feeding Tube 8fr 75cm
NGP8/85L	ENFit Nasogastric Feeding Tube 8fr 85cm
NGP8/85LW	ENFit Nasogastric Feeding Tube Weighted 8fr 85cm
NJP6/120L	ENFit Nasojejunal Feeding Tube 6Fr 120cm
NJP8/120L	ENFit Nasojejunal Feeding Tube 8Fr 120cm
NJP8/130L	ENFit Nasojejunal Feeding Tube 8Fr 130cm
NJP10/120L	ENFit Nasojejunal Feeding Tube 10Fr 120cm
NJP10/130L	ENFit Nasojejunal Feeding Tube 10Fr 130cm
NJP12/120L	ENFit Nasojejunal Feeding Tube 12Fr 120cm
NJP12/130L	ENFit Nasojejunal Feeding Tube 12Fr 130cm
NJP14/120L	ENFit Nasojejunal Feeding Tube 14Fr 120cm
NJP14/130L	ENFit Nasojejunal Feeding Tube 14Fr 130cm
NJP8/120LW	ENFit Nasojejunal Feeding Tube Weighted 8Fr 120cm
NJP10/120LW	ENFit Nasojejunal Feeding Tube Weighted 10Fr 120cm
NJP10/130LW	ENFit Nasojejunal Feeding Tube Weighted 10Fr 130cm
NJP12/120LW	ENFit Nasojejunal Feeding Tube Weighted 12Fr 120cm
NJP12/130LW	ENFit Nasojejunal Feeding Tube Weighted 12Fr 130cm
NJP14/120LW	ENFit Nasojejunal Feeding Tube Weighted 14Fr 120cm
NJP14/130LW	ENFit Nasojejunal Feeding Tube Weighted 14Fr 130cm
NJP16/120LW	ENFit Nasojejunal Feeding Tube Weighted 16Fr 120cm

Registered Office

Units 2-4, Rivington View Business Park, Station Road, Blackrod, Bolton BL6 5BN
Tel: 01204 695050 Fax: 01204 697755 email: info@medicina.co.uk www.medicina.co.uk

Medicina Ltd. A Company registered in England and Wales Reg. No. 02654669





Leaders in enteral delivery



Corrective Action

After a review and update of our Risk Management processes, both Medicina and HMC have updated our instructions for use to include the below explicit contraindication.

This device should not be used for high volume decompression in emergency situations, those at high risk of aspiration and/ or for decompression for high volume or high viscous fluid.



Action to be taken

If any of these products are identified within your trust/organisation, we advise;

- Please carefully read this Field Safety notice
- Please distribute throughout your organisations to relevant personnel.
- If you have any questions or queries in relation to this notice, please use the below means of communication to contact us.

Contact information regarding FSN

Shaista Patel

QA/RA Administrator

Unit 2, Rivington View Business Park, Station Road, Blackrod, Bolton

BL6 5BN

Email: quality.administrator@medicina.co.uk Tel: 01204 695050

Please confirm you have received this notice by completing the attached acknowledgement form and return the completed section by post or email to us as soon as possible.

Note: Medicina confirms the Medicines and Healthcare products Regulatory Agency (MHRA) have been provided with a copy of this Notice and updated Instructions for Use.

Yours Sincerely,



Caterina Buono
Associate QA/RA Manager
Medicina Ltd.



Danilo Bosetti
QA/RA Manger
HMC Premedical S.p.A





Leaders in enteral delivery



Acknowledgement Form of Field Safety Notice – September 2021

Please email: Shaista Patel, Quality Assurance Administrator

Email: Quality.administrator@medicina.co.uk

We confirm receipt of this Field Safety Notice – FSN006 and that this is received, read, and distributed to anyone it may concern within our organisation.

Trust/Organisation Name:	
Department:	
Representative Name and signature:	
Date:	

