Ultra-PEP

PEP Therapy

- | | | | |
|-----------|---|--------------|--|
| EN | ULTRA-PEP for PEP (positive expiratory pressure) therapy | NO | ULTRA-PEP for PEP (positiv ekspirasjonstrykk) behandling |
| AR | ULTRA-PEP لعلاج ضغط الزفير الإيجابي (PEP) | PL | ULTRA-PEP dla terapii PEP (dodatkim ciśnieniem wydechowym) |
| DE | ULTRA-PEP für PEP-Therapie (positiver expiratorischer Druck) | PT | ULTRA-PEP para terapia PEP (pressão expiratória positiva) |
| ES | ULTRA-PEP para la terapia PEP (presión espiratoria positiva) | PT-BR | ULTRA-PEP para terapia PEP (pressão expiratória positiva) |
| FI | ULTRA-PEP PEP-hoitoon (eli positiivinen uloshengityspaine) | RO | ULTRA-PEP pentru terapia PEP (presiune expiratorie pozitivă) |
| FR | ULTRA-PEP pour thérapie à PEP (pression expiratoire positive) | RU | ULTRA-PEP для PEP-терапии (положительное давление на выдохе) |
| HU | ULTRA-PEP PEP (pozitív kilégzési nyomás) terápiához | SR | ULTRA-PEP za PEP (pozitivni izdisajni pritisak) terapiju |
| IT | ULTRA-PEP per terapia PEP (pressione espiratoria positiva) | SV | ULTRA-PEP för PEP-behandling (positivt utandningstryck) |
| JA | PEP (呼吸陽圧) 療服用 ULTRA-PEP | TH | ULTRA-PEP สำหรับการบำบัดด้วยวิธี PEP (แรงดันหายใจด้านบวก) |
| KO | PEP (호기양압) 치료용 ULTRA-PEP | TL | ULTRA-PEP para sa PEP (positibong pressure ng papalabas na hangin) therapy |
| NL | ULTRA-PEP voor PEP (positieve eindexpiratoire druk)-therapie | TR | PEP (pozitif ekspiratuvar basınc) tedavisi için ULTRA-PEP |

EN	(English)	ULTRA-PEP for PEP (positive expiratory pressure) therapy	2
AR	(العربية)	ULTRA-PEP لعلاج ضغط الزفير الإيجابي (P)	3
DE	(Deutsch)	ULTRA-PEP für PEP-Therapie (positiver expiratorischer Druck)	4
ES	(Español)	ULTRA-PEP para la terapia PEP (presión espiratoria positiva)	5
FI	(Suomi)	ULTRA-PEP PEP-hoitoon (eli positiivinen uloshengityspaine)	6
FR	(Français)	ULTRA-PEP pour thérapie à PEP (pression expiratoire positive)	7
HU	(Magyar)	ULTRA-PEP PEP (pozitív kilégzési nyomás) terápiához	8
IT	(Italiano)	ULTRA-PEP per terapia PEP (pressione espiratoria positiva)	9
JA	(日本語)	PEP (呼吸陽圧) 療法用 ULTRA-PEP	10
KO	(한국어)	PEP (호기양압) 치료용 ULTRA-PEP	11
NL	(Nederlands)	ULTRA-PEP voor PEP (positieve eindexpiratoire druk)-therapie	12
NO	(Norsk)	ULTRA-PEP for PEP (positivt ekspirasjonstrykk) behandling	13
PL	(Język polski)	ULTRA-PEP dla terapii PEP (dodatnim ciśnieniem wydechowym)	14
PT	(Português)	ULTRA-PEP para terapia PEP (pressão expiratória positiva)	15
PT-BR	(Português Brasil)	ULTRA-PEP para terapia PEP (pressão expiratória positiva)	16
RO	(Română)	ULTRA-PEP pentru terapia PEP (presiune expiratorie pozitivă)	17
RU	(Русский)	ULTRA-PEP для PEP-терапии (положительное давление на выдохе)	18
SR	(Srpski)	ULTRA-PEP za PEP (pozitivni izdisajni pritisak) terapiju	19
SV	(Svenska)	ULTRA-PEP för PEP-behandling (positivt utandningstryck)	20
TH	(ไทย)	ULTRA-PEP สำหรับการบำบัดด้วยวิธี PEP (แรงดันหายใจด้านบวก)	21
TL	(Tagalog)	ULTRA-PEP para sa PEP (positibong pressure ng papalabas na hangin) therapy	22
TR	(Türkçe)	PEP (pozitif ekspiratuvar basıncı) tedavisi için ULTRA-PEP	23

for PEP (positive expiratory pressure) therapy

Key to Symbols:

Patient End

Adjust length if
requiredDetachable
Connection

Instructions For Use

1. This product is warranted for use on 1 (one) patient for a maximum period of 28-days from date of initial use, subject to cleaning not less than once per week.
2. Remove product from primary packaging and inspect contents for damage, ensuring that no separate components become mislaid.
3. Complete any final assembly steps and, if required, add additional components supplied separately.
4. The patient should be seated and upright.
5. Patient to follow these steps:
 - a. Adjust the expiratory valve cap to a position between 1 and 4 (and moderate as necessary thereafter).
 - b. Put the mouthpiece in your mouth and seal your lips around it.
 - c. Make a normal inhalation and then exhale slowly but firmly against the resistance. Try to breathe all the way out to the end of your breath. Exhalation should last 6 seconds. You should be able to hear a hissing sound as the air comes out of the valve.
 - d. Repeat for 10 breaths if possible. You may need to cough before you reach 10th breath. If so, take the device out of your mouth and cough to clear any loosened secretions.
 - e. Rest for 30 seconds.
 - f. Cough and clear any phlegm that has been loosened and rest again.
 - g. Repeat the above cycle at least 3 times or as often as is necessary to clear the chest.
6. The device can be used as often as you wish (or as advised by a caregiver). A morning and evening session is often sufficient.
7. Ensure that use ceases within 28-days from date of initial use.

Warnings

1. Keep package sealed until use – risk of ingestion or inhalation of small objects and blockage of gas paths.
2. For cleaning, take the device apart and wash the parts in soapy water (maximum 50°C) and allow to fully air dry before re-assembly and further use.

Information

Device Name	PEP Therapy Device
Intended Use	For generation of resistance to passage of gas during expiratory breaths
Indications for Use	For secretion mobilisation in post-operative patients, cystic fibrosis patients and patients indicated for respiratory physiotherapy
Contraindications	Those patients not medically-indicated for the intended therapy. Breathless patients and weak patients
Warnings	Refer to any listed warnings
Intended Users	Appropriately trained respiratory physiotherapists
Pre-use Precautions	Refer to Instructions For Use
Performance Characteristics	Refer to any listed performance characteristics
Disposal	Dispose as per the hospital's protocol for management of contaminated clinical waste. If used in a home setting, dispose as general waste for incineration
Device Malfunction	In the event of a device malfunction or an unexpected change in device performance, cease use of the device immediately
Serious Adverse Incidents	In the event of a malfunction of the device when in use on a patient and leading to a Serious Adverse Incident, contact the device manufacturer and the relevant National Competent Authority. (A Serious Adverse Incident is considered to be the death or serious deterioration in the state of health of the patient or user).
Use Restrictions	Use only with compatible devices. Consult Armstrong Medical for confirmation of compatible devices.
Other Information	None provided

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