

Urgent Field Safety Notice

Rociale Healthcare Limited

Various Procedure Packs Containing 1ml and 1.5ml ChloroPrep

Date: 04/06/2026

FSN 2026-01

Details of affected devices:

Becton Dickinson UK Ltd (BD) has issued DMRC - 39617055 for a Class 2 recall of ChloroPrep 2% w/v / 70% v/v cutaneous solution 1mL applicators and ChloroPrep Frepp 2% w/v / 70% v/v cutaneous solution 1.5 mL applicators, with an expiry date up to and including February 2028. Rociale Healthcare has incorporated these medicinal products into the procedure packs listed in Appendix 1, all of which are affected by this Defective Medicine:

Description of the problem:

Becton Dickinson UK Ltd are recalling batches of ChloroPrep 2% w/v / 70% v/v cutaneous solution 1mL applicators and ChloroPrep Frepp 2% w/v / 70% v/v cutaneous solution 1.5 mL applicators due to a potential breach of sterility in the packaging process. The recall applies to all batches with an expiry date up to and including 02/2028. Batches with an expiry date after 02/2028 are not impacted.

This defect could increase the risk of the applicator device being contaminated with pathogens, which could lead to increased infection rates for the patients.

Action to be taken by the user:

Rociale are kindly requesting that customers remove the ChloroPrep from the affected pack and discard in accordance with local medicinal product handling guidelines. Please log this issue on the Rociale customer complaint portal here: <https://www.rocialeacutecare.com/report-a-problem/>, advising us of the quantity affected and confirming that the ChloroPrep has been discarded. If you would prefer to return the pack for a full credit, please let us know on the Portal after which our customer services team will organise the uplift and replacement.

Transmission of this Field Safety Notice: (if appropriate)

This notice needs to be passed on all those who need to be aware within your organisation, or to any organisation where the potentially affected products may have been transferred (if appropriate).

Please transfer this notice to other organisations on which this action has an impact, if appropriate.

Please maintain awareness on this notice and resulting action for an appropriate period to ensure effectiveness of the corrective action, if appropriate.

Contact reference team:

If you have any queries regarding this Field Safety Notice, please contact:

Rociale Healthcare Limited
Regulatory Team
Ty Mynydd
Cwm Cynon Business Park (North)
Mountain Ash
Rhondda Cynon Taff
CF45 4ER

Email: regulatory@rociallehealthcare.com

Tel: 01443 471300

Pack Ref	Description	WO number	Pack expiry date
RML506-040	(DP)AW PVC Cannulation Pack	W543608	2026-07
RML500-001	(DP) IV Cannulation Pack	W543649	2026-07
RML506-075	(DP)Adult Cannulation Pack	W543648	2026-07
RML506-040	(DP)AW PVC Cannulation Pack	W543753	2026-07
RMT0990	(DP) Arterial pack	W543891	2026-12
RML506-075	(DP)Adult Cannulation Pack	W544443	2026-08
RML506-075	(DP)Adult Cannulation Pack	W544582	2026-08
RML500-001	(DP) IV Cannulation Pack	W544632	2026-10
RML506-040	(DP)AW PVC Cannulation Pack	W544634	2026-10
RML506-040	(DP)AW PVC Cannulation Pack	W544766	2026-10
RML506-118	(DP) Adult Cannulation Pack	W544808	2026-08
RML506-040	(DP)AW PVC Cannulation Pack	W544912	2026-10
RML506-040	(DP)AW PVC Cannulation Pack	W545139	2026-11
RMT0990	(DP) Arterial pack	W545183	2026-11
RML506-075	(DP)Adult Cannulation Pack	W545185	2026-08
RML506-040	(DP)AW PVC Cannulation Pack	W545286	2026-11
RML506-153	(DP) IV Pack	W545377	2026-08
RML500-001	(DP) IV Cannulation Pack	W545440	2026-09
RML506-118	(DP) Adult Cannulation Pack	W545441	2026-08
RML506-040	(DP)AW PVC Cannulation Pack	W545486	2026-11
RML506-040	(DP)AW PVC Cannulation Pack	W545579	2026-10
RML506-040	(DP)AW PVC Cannulation Pack	W545818	2026-11
RML506-040	(DP)AW PVC Cannulation Pack	W545831	2026-11
RML506-040	(DP)AW PVC Cannulation Pack	W545983	2026-11
RML506-118	(DP) Adult Cannulation Pack	W546066	2026-12
RML506-040	(DP)AW PVC Cannulation Pack	W546258	2026-11
RML506-040	(DP)AW PVC Cannulation Pack	W546380	2026-12
RML506-040	(DP)AW PVC Cannulation Pack	W546556	2026-12
RML506-118	(DP) Adult Cannulation Pack	W546557	2026-11
RML506-040	(DP)AW PVC Cannulation Pack	W546613	2027-01
RML102-225	(DP) Adult IV Insertion Pack	W546696	2027-01
RML102-225	(DP) Adult IV Insertion Pack	W547276	2026-08
RML506-040	(DP)AW PVC Cannulation Pack	W547277	2026-08
RML112-193	(DP) Cannulation Pack	W547281	2026-08
RML506-040	(DP)AW PVC Cannulation Pack	W547449	2027-01
RML506-166	(DP)IV Pack Option 2 with leaflet	W547278	2026-08
RML506-040	(DP)AW PVC Cannulation Pack	W547493	2027-02
RML506-075	(DP)Adult Cannulation Pack	W547491	2026-11
RML506-040	(DP)AW PVC Cannulation Pack	W547631	2027-02
RML506-040	(DP)AW PVC Cannulation Pack	W547677	2027-02
RML506-040	(DP)AW PVC Cannulation Pack	W547738	2027-02
RML506-118	(DP) Adult Cannulation Pack	W547736	2026-11
RML506-040	(DP)AW PVC Cannulation Pack	W547793	2027-02
RML102-225	(DP) Adult IV Insertion Pack	W547790	2026-11
RML506-040	(DP)AW PVC Cannulation Pack	W547866	2027-02
RML506-166	(DP)IV Pack Option 2 with leaflet	W547865	2027-02
RML102-225	(DP) Adult IV Insertion Pack	W547838	2027-02
RML500-001	(DP) IV Cannulation Pack	W547864	2027-02
RML506-153	(DP) IV Pack	W547964	2026-08
RML112-193	(DP) Cannulation Pack	W548064	2027-01

Pack Ref	Description	WO number	Pack expiry date
RML506-118	(DP) Adult Cannulation Pack	W548065	2026-11
RML506-040	(DP)AW PVC Cannulation Pack	W548066	2027-01
RML102-225	(DP) Adult IV Insertion Pack	W548108	2027-01
RML506-040	(DP)AW PVC Cannulation Pack	W548237	2027-01
RML506-118	(DP) Adult Cannulation Pack	W548325	2026-11
RML112-193	(DP) Cannulation Pack	W548375	2027-02
RML500-001	(DP) IV Cannulation Pack	W548377	2027-02
RML506-075	(DP)Adult Cannulation Pack	W548482	2026-11
RML506-040	(DP)AW PVC Cannulation Pack	W548530	2027-02
MTK1046-ISK-20B	(DP) Cannulation Pack - UK	W548535	2027-02
RML506-166	(DP)IV Pack Option 2 with leaflet	W548653	2027-02
RML506-075	(DP)Adult Cannulation Pack	W548654	2026-11
RML506-040	(DP)AW PVC Cannulation Pack	W548690	2027-02
RML506-118	(DP) Adult Cannulation Pack	W548748	2026-11
RML102-225	(DP) Adult IV Insertion Pack	W548803	2026-11
RML102-225	(DP) Adult IV Insertion Pack	W548886	2027-03
RML506-040	(DP)AW PVC Cannulation Pack	W548957	2027-02
RMT0990	(DP) Arterial pack	W549015	2027-02
RML506-040	(DP)AW PVC Cannulation Pack	W549121	2026-11
RML506-118	(DP) Adult Cannulation Pack	W549178	2026-11
RML506-040	(DP)AW PVC Cannulation Pack	W549205	2027-03
RML112-193	(DP) Cannulation Pack	W549250	2027-03
RML506-040	(DP)AW PVC Cannulation Pack	W549315	2027-02
RML506-040	(DP)AW PVC Cannulation Pack	W549472	2027-02
RML506-040	(DP)AW PVC Cannulation Pack	W549536	2027-02
RML506-075	(DP)Adult Cannulation Pack	W549660	2026-11
RML506-118	(DP) Adult Cannulation Pack	W549671	2026-11
RML500-001	(DP) IV Cannulation Pack	W549718	2027-02
RML506-040	(DP)AW PVC Cannulation Pack	W549717	2027-01
RML506-153	(DP) IV Pack	W549776	2026-08
RML506-118	(DP) Adult Cannulation Pack	W549915	2026-11
RML506-040	(DP)AW PVC Cannulation Pack	W549967	2027-02
RML506-153	(DP) IV Pack	W549966	2026-08
RMT0990	(DP) Arterial pack	W549965	2027-02
RML506-118	(DP) Adult Cannulation Pack	W550321	2026-11
RML102-225	(DP) Adult IV Insertion Pack	W551022	2027-04
RML102-225	(DP) Adult IV Insertion Pack	W551097	2027-04
RML506-118	(DP) Adult Cannulation Pack	W550897	2026-11
RML506-040	(DP)AW PVC Cannulation Pack	W551109	2027-03
MTK1046-ISK17K	(DP) Blood Culture Kit, w Kurin, for BD Bottles	W551209	2026-07
RML506-040	(DP)AW PVC Cannulation Pack	W551325	2027-03
MTK1046-ISK17K	(DP) Blood Culture Kit, w Kurin, for BD Bottles	W551326	2026-07
RML506-118	(DP) Adult Cannulation Pack	W551373	2027-04
RML506-075	(DP)Adult Cannulation Pack	W551369	2027-04
RML506-040	(DP)AW PVC Cannulation Pack	W551432	2027-04

Pack Ref	Description	WO number	Pack expiry date
RML506-040	(DP)AW PVC Cannulation Pack	W551485	2027-04
RML102-225	(DP) Adult IV Insertion Pack	W551519	2027-04
RML506-040	(DP)AW PVC Cannulation Pack	W551559	2027-04
RML506-166	(DP)IV Pack Option 2 with leaflet	W551596	2027-04
RML506-075	(DP)Adult Cannulation Pack	W551633	2027-04
RML506-251	(DP) Blood Culture Pack	W551679	2026-06
RML102-225	(DP) Adult IV Insertion Pack	W551724	2027-04
RML506-040	(DP)AW PVC Cannulation Pack	W551783	2027-05
RML506-118	(DP) Adult Cannulation Pack	W551828	2027-04
RML500-001	(DP) IV Cannulation Pack	W551827	2027-04
RML506-166	(DP)IV Pack Option 2 with leaflet	W551825	2027-04
RML506-040	(DP)AW PVC Cannulation Pack	W551894	2027-05
RML506-040	(DP)AW PVC Cannulation Pack	W551918	2027-05
RML506-040	(DP)AW PVC Cannulation Pack	W551964	2027-05
RML506-040	(DP)AW PVC Cannulation Pack	W552003	2026-11
RML506-040	(DP)AW PVC Cannulation Pack	W552098	2027-05
RML506-251	(DP) Blood Culture Pack	W552142	2026-07
RML506-118	(DP) Adult Cannulation Pack	W552140	2027-04
RML506-040	(DP)AW PVC Cannulation Pack	W552177	2027-05
RML101-223	(DP) Blood Culture pack	W552220	2027-05
RML506-040	(DP)AW PVC Cannulation Pack	W552261	2027-05
RML506-040	(DP)AW PVC Cannulation Pack	W552336	2027-05
RML102-225	(DP) Adult IV Insertion Pack	W552223	2027-05
RML506-118	(DP) Adult Cannulation Pack	W552378	2027-04
RML506-040	(DP)AW PVC Cannulation Pack	W552417	2027-05
RML500-001	(DP) IV Cannulation Pack	W552457	2027-05
RML101-223	(DP) Blood Culture pack	W552458	2027-07
RML506-240	(DP) AW PVC Cannulation Procedure Pack	W552460	2027-05
RML506-239	(DP) AW Blood Culture Procedure Pack	W552459	2026-06
RML506-044	(DP) Blood Culture Pack	W552506	2026-06
RML506-075	(DP)Adult Cannulation Pack	W552507	2027-04
RML506-040	(DP)AW PVC Cannulation Pack	W552545	2027-05
RMT0990	(DP) Arterial pack	W552636	2027-06
RML506-040	(DP)AW PVC Cannulation Pack	W552638	2027-04
RML506-118	(DP) Adult Cannulation Pack	W552550	2027-04
MTK1046-ISK17K	(DP) Blood Culture Kit, w Kurin, for BD Bottles	W552689	2026-07
RML506-044	(DP) Blood Culture Pack	W552699	2026-06
MTK1046-ISK-20B	(DP) Cannulation Pack - UK	W552686	2026-12
RML102-225	(DP) Adult IV Insertion Pack	W552796	2027-04
RML506-153	(DP) IV Pack	W552856	2026-08
RML506-040	(DP)AW PVC Cannulation Pack	W552814	2027-04
RML506-250	(DP) AW Paediatric Blood Culture Procedure Pack	W552871	2026-09
RML506-040	(DP)AW PVC Cannulation Pack	W553008	2027-04

Pack Ref	Description	WO number	Pack expiry date
RML506-139	(DP) AW Blood Culture Pack	W553068	2026-06
MTK1046-ISK17K	(DP) Blood Culture Kit, w Kurin, for BD Bottles	W553066	2026-08
RML101-223	(DP) Blood Culture pack	W553116	2027-06
RML506-118	(DP) Adult Cannulation Pack	W553200	2027-04
RML506-044	(DP) Blood Culture Pack	W553199	2026-06
RML506-040	(DP)AW PVC Cannulation Pack	W553320	2027-05
RML101-366	(DP)Blood Culture pack	W553487	2026-08
RML506-040	(DP)AW PVC Cannulation Pack	W553488	2027-05
RML506-075	(DP)Adult Cannulation Pack	W553511	2027-04
RML102-225	(DP) Adult IV Insertion Pack	W553651	2027-08
RML102-225	(DP) Adult IV Insertion Pack	W553684	2027-08
MTK1046-ISK23	(DP) Blood Culture Kit	W553691	2026-09
RML506-166	(DP)IV Pack Option 2 with leaflet	W553722	2027-05
RML506-251	(DP) Blood Culture Pack	W553753	2026-09
RML506-075	(DP)Adult Cannulation Pack	W553794	2027-04
RML506-118	(DP) Adult Cannulation Pack	W553853	2027-04
RML506-040	(DP)AW PVC Cannulation Pack	W553854	2027-05
RML506-040	(DP)AW PVC Cannulation Pack	W553898	2027-05
RML101-220	(DP)Adult Peripheral IV Cannulation Pa	W553944	2026-06
MTK1046-ISK17K	(DP) Blood Culture Kit, w Kurin, for BD Bottles	W553980	2026-07
RML506-040	(DP)AW PVC Cannulation Pack	W554018	2027-05
MTK1046/ISK-18	(DP) Blood Culture Kit	W554022	2027-03
RML102-225	(DP) Adult IV Insertion Pack	W554141	2027-08
RML506-040	(DP)AW PVC Cannulation Pack	W554107	2027-05
RML506-019	(DP)Peripheral Blood Culture Pack	W554106	2026-06
RML102-225	(DP) Adult IV Insertion Pack	W554065	2027-08
RML506-139	(DP) AW Blood Culture Pack	W554064	2026-08
RML506-141	(DP) Cannulation Pack	W554262	2026-09
RML506-040	(DP)AW PVC Cannulation Pack	W554265	2027-05
RML506-133	(DP) Blood Culture Pack	W554361	2026-06
RML506-118	(DP) Adult Cannulation Pack	W554402	2027-04
RML500-001	(DP) IV Cannulation Pack	W554401	2027-05
RML506-040	(DP)AW PVC Cannulation Pack	W554488	2027-05
RML506-251	(DP) Blood Culture Pack	W554530	2026-09
RML506-040	(DP)AW PVC Cannulation Pack	W554568	2027-05
MTK1046/ISK-18	(DP) Blood Culture Kit	W554572	2027-06
RML506-040	(DP)AW PVC Cannulation Pack	W554711	2027-05
RML506-118	(DP) Adult Cannulation Pack	W554762	2027-04
RML506-139	(DP) AW Blood Culture Pack	W554802	2026-08
RML506-040	(DP)AW PVC Cannulation Pack	W554916	2027-05
RML506-118	(DP) Adult Cannulation Pack	W554965	2027-04
RML506-040	(DP)AW PVC Cannulation Pack	W555011	2027-05
RML506-044	(DP) Blood Culture Pack	W555010	2026-08
RML506-040	(DP)AW PVC Cannulation Pack	W555052	2027-05
RML506-075	(DP)Adult Cannulation Pack	W555050	2027-04

Pack Ref	Description	WO number	Pack expiry date
RML506-019	(DP)Peripheral Blood Culture Pack	W555092	2026-06
RML101-220	(DP)Adult Peripheral IV Cannulation Pa	W555136	2026-07
MTK1046-ISK23	(DP) Blood Culture Kit	W555170	2026-08
RML506-044	(DP) Blood Culture Pack	W555213	2026-08
RML101-366	(DP)Blood Culture pack	W555244	2026-08
RMT0990	(DP) Arterial pack	W555367	2027-08
RML506-040	(DP)AW PVC Cannulation Pack	W555617	2027-09
RML102-225	(DP) Adult IV Insertion Pack	W555688	2027-09
RML506-019	(DP)Peripheral Blood Culture Pack	W555724	2026-06
RML506-040	(DP)AW PVC Cannulation Pack	W555777	2027-09
RML000-400	(DP) Blood Culture Pack	W555728	2026-06
RML506-133	(DP) Blood Culture Pack	W555918	2026-07
RML506-040	(DP)AW PVC Cannulation Pack	W555899	2027-09
RML506-118	(DP) Adult Cannulation Pack	W555898	2027-04
MTK1046-ISK23	(DP) Blood Culture Kit	W555896	2026-10
RML506-251	(DP) Blood Culture Pack	W556030	2026-10
RML101-223	(DP) Blood Culture pack	W556088	2027-09
RML506-040	(DP)AW PVC Cannulation Pack	W556126	2027-09
RML102-225	(DP) Adult IV Insertion Pack	W556162	2027-09
RML506-040	(DP)AW PVC Cannulation Pack	W556210	2027-09
RML506-141	(DP) Cannulation Pack	W556209	2026-10
RML506-166	(DP)IV Pack Option 2 with leaflet	W556286	2027-08
RML506-075	(DP)Adult Cannulation Pack	W556290	2027-04
RML506-118	(DP) Adult Cannulation Pack	W556337	2027-04
MTK1046-ISK-20B	(DP) Cannulation Pack - UK	W556334	2027-08
RML102-225	(DP) Adult IV Insertion Pack	W556384	2027-10
THR01-25-080	(DP) ANTT Cannula Pack - Double Lumen	W556509	2027-10
THR01-25-080(1)	(DP) ANTT Cannula Pack - Single Lumen	W556510	2027-10
RML112-193	(DP) Cannulation Pack	W556426	2027-09
RML506-139	(DP) AW Blood Culture Pack	W556436	2026-08
MTK1046-ISK-20B	(DP) Cannulation Pack - UK	W556564	2027-08
RML506-019	(DP)Peripheral Blood Culture Pack	W556565	2026-06
RML500-001	(DP) IV Cannulation Pack	W556566	2027-09
RML506-251	(DP) Blood Culture Pack	W556614	2026-12
RML506-040	(DP)AW PVC Cannulation Pack	W556615	2027-09
RML506-166	(DP)IV Pack Option 2 with leaflet	W556660	2027-09
RML506-040	(DP)AW PVC Cannulation Pack	W556781	2027-09
RML506-044	(DP) Blood Culture Pack	W556740	2026-08
RML506-118	(DP) Adult Cannulation Pack	W556830	2027-04
RML506-251	(DP) Blood Culture Pack	W556827	2026-12
RML506-040	(DP)AW PVC Cannulation Pack	W556957	2027-09
RML102-225	(DP) Adult IV Insertion Pack	W556987	2027-10
RML101-220	(DP)Adult Peripheral IV Cannulation Pa	W557028	2026-07
RML506-189	(DP) Paediatric IV Pack	W557026	2027-09
RML506-040	(DP)AW PVC Cannulation Pack	W557060	2027-09

Pack Ref	Description	WO number	Pack expiry date
RML506-139	(DP) AW Blood Culture Pack	W557106	2026-08
RML506-153	(DP) IV Pack	W557150	2026-08
RML506-118	(DP) Adult Cannulation Pack	W557210	2027-04
RML102-225	(DP) Adult IV Insertion Pack	W557247	2027-11
RML506-040	(DP)AW PVC Cannulation Pack	W557287	2027-09
RML506-040	(DP)AW PVC Cannulation Pack	W557356	2027-11
RML112-193	(DP) Cannulation Pack	W557392	2027-11
RML102-225	(DP) Adult IV Insertion Pack	W557486	2027-11
RML102-225	(DP) Adult IV Insertion Pack	W557556	2027-09
MTK1046/ISK-18	(DP) Blood Culture Kit	W557651	2027-06
RML101-366	(DP)Blood Culture pack	W557748	2026-09
RML506-040	(DP)AW PVC Cannulation Pack	W557749	2027-11
RML506-040	(DP)AW PVC Cannulation Pack	W557697	2027-11
RML101-220	(DP)Adult Peripheral IV Cannulation Pa	W557792	2026-11
RML506-133	(DP) Blood Culture Pack	W557791	2026-09
RML506-075	(DP)Adult Cannulation Pack	W557871	2027-08
RML506-118	(DP) Adult Cannulation Pack	W557873	2027-11
RML506-141	(DP) Cannulation Pack	W557872	2027-01
RML506-040	(DP)AW PVC Cannulation Pack	W557919	2027-11
RML506-040	(DP)AW PVC Cannulation Pack	W557954	2027-11
RML500-001	(DP) IV Cannulation Pack	W557981	2027-11
RML506-040	(DP)AW PVC Cannulation Pack	W558015	2027-11
RML506-139	(DP) AW Blood Culture Pack	W558105	2026-09
RML506-118	(DP) Adult Cannulation Pack	W558180	2027-11
RML506-040	(DP)AW PVC Cannulation Pack	W558218	2027-08
RML506-040	(DP)AW PVC Cannulation Pack	W558293	2027-08
RML102-212	(DP) Paediatric IV Pack	W558394	2027-11
MTK1046-ISK-20B	(DP) Cannulation Pack - UK	W558473	2027-12
RML101-220	(DP)Adult Peripheral IV Cannulation Pa	W558476	2026-11
RML101-220	(DP)Adult Peripheral IV Cannulation Pa	W558610	2026-11
RML506-044	(DP) Blood Culture Pack	W558780	2026-10
RML506-251	(DP) Blood Culture Pack	W558781	2027-01
RML506-075	(DP)Adult Cannulation Pack	W558825	2027-09
RML506-139	(DP) AW Blood Culture Pack	W558979	2027-01
RML506-133	(DP) Blood Culture Pack	W559145	2027-02
RML506-157	(DP) Cannulation Pack	W559233	2027-11
MTK1046-ISK17K	(DP) Blood Culture Kit, w Kurin, for BD Bottles	W559479	2027-05
RML506-044	(DP) Blood Culture Pack	W559480	2027-01
RML506-133	(DP) Blood Culture Pack	W559526	2027-02
RML101-220	(DP)Adult Peripheral IV Cannulation Pa	W559529	2026-11
RML506-075	(DP)Adult Cannulation Pack	W559645	2027-09
RML506-040	(DP)AW PVC Cannulation Pack	W559735	2027-12
RML506-040	(DP)AW PVC Cannulation Pack	W559769	2027-12
RML506-166	(DP)IV Pack Option 2 with leaflet	W559810	2027-12
RML102-225	(DP) Adult IV Insertion Pack	W559809	2027-12

Pack Ref	Description	WO number	Pack expiry date
RML506-040	(DP)AW PVC Cannulation Pack	W559896	2027-12
RML506-040	(DP)AW PVC Cannulation Pack	W559941	2027-12
RML102-225	(DP) Adult IV Insertion Pack	W559945	2027-12
RML101-220	(DP)Adult Peripheral IV Cannulation Pa	W559946	2026-11
RML506-153	(DP) IV Pack	W559944	2027-12
RML506-139	(DP) AW Blood Culture Pack	W560128	2027-01
RMT1698	(DP)Blood Culture Pack	W560083	2027-02
RML506-075	(DP)Adult Cannulation Pack	W560082	2027-08
RML101-366	(DP)Blood Culture pack	W560322	2027-01
RML506-040	(DP)AW PVC Cannulation Pack	W560368	2027-10
RML506-075	(DP)Adult Cannulation Pack	W560324	2027-11
RML506-040	(DP)AW PVC Cannulation Pack	W560439	2027-10
RML112-193	(DP) Cannulation Pack	W560370	2027-10
RML506-251	(DP) Blood Culture Pack	W560491	2027-03
RML506-044	(DP) Blood Culture Pack	W560490	2027-01
RML506-075	(DP)Adult Cannulation Pack	W560920	2027-11
RML506-139	(DP) AW Blood Culture Pack	W560971	2027-02
RML101-220	(DP)Adult Peripheral IV Cannulation Pa	W560972	2026-11
RML506-044	(DP) Blood Culture Pack	W561067	2027-02
RML506-044	(DP) Blood Culture Pack	W561241	2027-02
MTK1046-ISK17K	(DP) Blood Culture Kit, w Kurin, for BD Bottles	W561433	2027-07
RMT0990	(DP) Arterial pack	W561536	2027-08
RML506-040	(DP)AW PVC Cannulation Pack	W561538	2028-04
RML506-139	(DP) AW Blood Culture Pack	W561856	2027-02
RML506-251	(DP) Blood Culture Pack	W562394	2027-03

Appendix 1