(en) Blood/Fluid Warming High Flow Set • With 48 inch extension and auto-venting bubble trap • Fluid warmed from 4°C inlet temperature to 33°C - 41°C, at flow rates from KVO to 167 mL/min • Fluid warmed from 20°C inlet temperature to 33°C - 41°C, at flow rates from KVO to 333 mL/min • Priming volume: 89 mL • Sterile and nonpyrogenic fluid pathway. Do not use if sterile barrier system has been breached. • For use only with a 3M™ Ranger™ blood/fluid warming unit model 245

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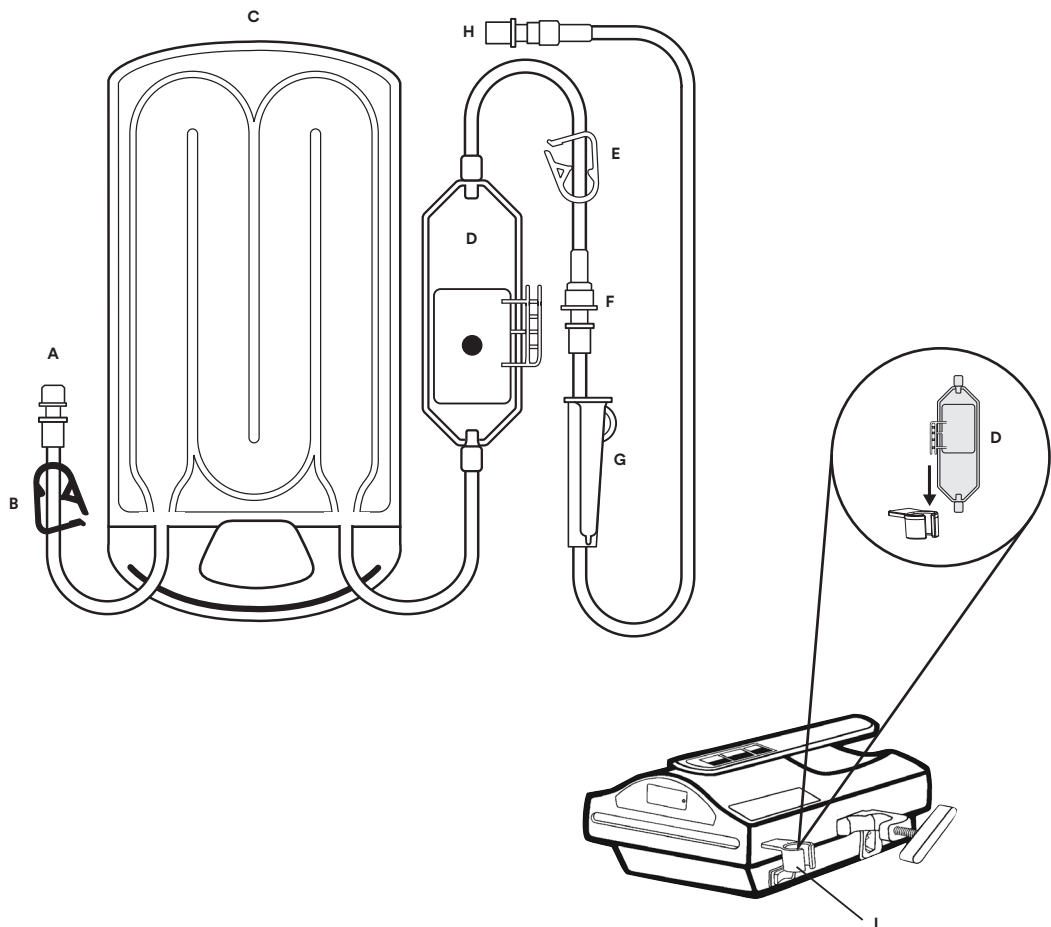
en Indications for Use

The Ranger blood/fluid warmer is intended to warm blood, blood products, and liquids.

Patient Population and Settings

Adult and pediatric patients being treated in operating rooms, emergency trauma settings, or other areas where blood/fluids are infused.

Explanation of Signal Word Consequences

WARNING: Indicates a hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION: Indicates a hazardous situation which, if not avoided, could result in minor or moderate injury.

WARNING: To reduce the risks associated with air embolism:

- Never infuse fluids if air bubbles are present in the fluid line
- Ensure all luer connections are tightened
- Do not mount the Ranger blood fluid warming system higher than patient level, as air embolism may result

WARNING: To reduce the risks of biohazard exposure, cross-contamination or infection:

- Do not reuse. This product is designed for single patient use only.
- Do not use if caps are not in place
- Ensure that all luer connections are securely tightened prior to priming set
- Do not exceed 300 mmHg pressure; ensure inline hand pressure pumps or other pumps do not exceed 300 mmHg pressure.

WARNING: To reduce the risks of leaking, slips and falls, and loss of therapy:

- Do not reuse. This product is designed for single patient use only.
- Ensure that all luer connections are securely tightened prior to priming set.

WARNING: To reduce the risks associated with potential blood loss:

- Do not use in combination with an extracorporeal circuit.

WARNING: To reduce the risk associated with patient exposure to leakage current that exceeds Type BF equipment limits:

- Do not use in combination with other medical equipment without verifying that the total leakage current from the combined equipment does not exceed the safety limits for Type BF equipment.

CAUTION: To reduce the risk of leaking and loss of therapy:

- Do not exceed 300 mmHg pressure; ensure inline hand pressure pumps or other pumps do not exceed 300 mmHg pressure.
- Do not remove warming set from warming unit while in use.

CAUTION: To reduce the risk associated with warming platelets, cryoprecipitate, or granulocyte suspensions:

- Follow the AABB Guidelines for the use of blood warming devices which caution against warming when administering platelets, cryoprecipitate, or granulocyte suspensions.

CAUTION: To reduce the risks associated with potential material toxicity:

- Do not use in combination with an extracorporeal circuit as the Ranger blood/fluid warming system is not designed for use with a circulating blood system; therefore, biocompatibility testing has not been completed to assess genotoxicity.

Instructions for Use

Do not use if package has been previously opened or damaged. Refer to the 3M™ Ranger™ blood/fluid warming unit model 245 Operator's Manual for set up and use of the warming system.

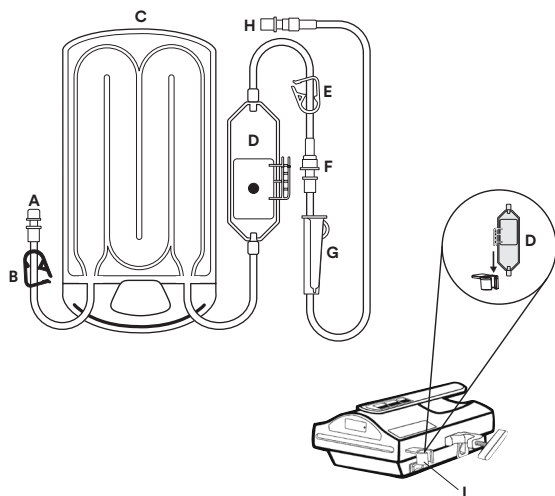
Figure 1: Illustration of Disposable set


- | | |
|-----------------------------|-----------------------|
| A) Inlet line | F) Luer connection |
| B) Blue inlet pinch clamp | G) Roller clamp |
| C) Fluid warming cassette | H) Patient connection |
| D) Auto-venting bubble trap | I) Bubble trap holder |
| E) White outlet pinch clamp | |

Figure 2

Peel To Open →

Peel To Open →

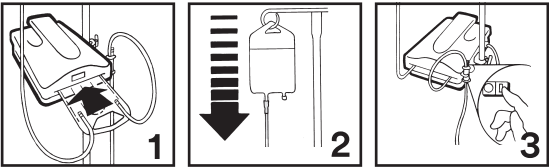Figure 3

Instructions

Set-up and Priming the disposable set

1. Adjust warming unit height to patient level or lower.
2. Open warming set pouch where indicated by the arrows (Fig. 3)
3. Before priming, slide the fluid warming cassette (Fig. 1, C) into the slot in the warming unit (Fig 3, #1).
4. Place the auto-venting bubble trap (Fig. 1, D) into the holder (Fi. 1, I) on the warming unit.
5. Tighten luer connections (Fig. 1, F)
6. Close clamp below the auto-venting bubble trap (Fig 1, E).
7. Close clamp above cassette (Fig. 1, B).
8. Connect the short inlet line (Fig. 1, A) on the Ranger warming set to the fluid source (primary IV administration set needed).
9. Open the clamp (Fig. 1, B) above cassette (Fig. 3, #2).
10. Allow the auto-venting bubble trap chamber to fill (Fig 1, E). The flow will stop when the chamber is properly filled. Gently tap the auto-venting bubble trap against the warming unit to expel any trapped air, if necessary.
11. Open the clamps below the auto-venting bubble trap and continue to prime the patient line.
12. Close any clamp to stop flow.
13. Turn the warming unit ON (Fig 3, #3).

The fluid warming set is now ready for use.

Removing Air from the Auto-venting bubble Trap

1. The auto-venting bubble trap is designed to continuously vent air from the patient line. Monitor fluid lines to ensure that they are air-free.
2. Gently tap the auto-venting bubble trap against the warming unit to expel any trapped air.

Removing the Disposable Set

1. Close the inlet clamp (Fig 1, B) and open all clamps distal to the cassette.
2. Disconnect the warming set from the fluid source (if applicable).
3. Allow fluid to flow into the patient (this may take 2 to 3 seconds) and close outlet clamp (F).
4. Disconnect patient line from disposable set.
5. Remove the fluid warming cassette from the warming unit and discard according to institutional protocol.

Symbol Glossary

Symbol Title	Symbol	Description and Reference
Manufacturer		Indicates the medical device manufacturer as defined in Medical Device Regulation (EU) 2017/745 formerly EU Directive 93/42/EEC. Source: ISO 15223, 5.1.1
Authorized Representative in European Community		Indicates the authorized representative in the European Community. Source: ISO 15223, 5.1.2, 2014/35/EU, and/or 2014/30/EU
Date of Manufacture		Indicates the date when the medical device was manufactured. Source: ISO 15223, 5.1.3
Use-by date		Indicates the date after which the medical device is not to be used. Source: ISO 15223, 5.1.4
Batch code		Indicates the manufacturer's batch code so that the batch or lot can be identified. Source : ISO 15223, 5.1.5
Catalogue number		Indicates the manufacturer's catalogue number so that the medical device can be identified. Source : ISO 15223, 5.1.6
Sterilized using ethylene oxide		Indicates a medical device that has been sterilized using ethylene oxide. Source: ISO 15223, 5.2.3
Do not use if package is damaged or open		Indicates a medical device that should not be used if the package has been damaged or opened. Source: ISO 15223, 5.2.8
Sterile fluid path		Indicates the presence of a sterile fluid path within the medical device in cases when other parts of the medical device, including the exterior, might not be supplied sterile. Source: ISO 15223, 5.2.9

Do not re-use		Indicates a medical device that is intended for one use or for use on a single patient during a single procedure. Source: ISO 15223, 5.4.2
Caution		Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself. Source: ISO 15223, 5.4.4
Natural rubber latex is not present		Indicates natural rubber or dry natural rubber latex is not present as a material of construction within the medical device or the packaging of a medical device. Source: ISO 15223, 5.4.5 and Annex B
Medical Device		Indicates the item is a medical device.
Unique device identifier		Indicates bar code to scan product information into patient electronic health record
Importer		Indicates the entity importing the medical device into the EU.
Single sterile barrier system		To identify a single layer that makes up the sterile barrier system (SBS) and differentiate it from layers of protective packaging designed to prevent damage to the sterile barrier system and its contents
CE Mark		Indicates conformity to all applicable European Union Regulations and Directives with notified body involvement.
Rx Only		Indicates that U.S. Federal Law restricts this device to sale by or on the order of a healthcare professional. 21 Code of Federal Regulations (CFR) sec. 801.109(b)(1)

For more information see, [HCBRegulatory.3M.com](https://www.hcbregulatory.3m.com)

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3M and Ranger are trademarks of 3M. Used under license in Canada.
© 2025, 3M. Unauthorized use prohibited. All rights reserved.
3M Deutschland GmbH